

Le dilateur anal est conçu pour favoriser à la fois le bien-être sexuel et l'exploration sensorielle.

Mode d'emploi

1. Lavez le dilateur à l'eau tiède avec le nettoyant Steri Clean avant et après chaque utilisation.
2. Appliquez un lubrifiant à base d'eau sur le dilateur et autour de la zone anale.
3. Installez-vous confortablement dans un endroit calme. Commencez par le plus petit format si vous utilisez un ensemble. Insérez doucement en effectuant de légers mouvements circulaires, sans forcer. Respirez profondément et laissez les muscles se détendre.
4. Laissez le dilateur en place quelques minutes, en le bougeant légèrement si vous le souhaitez. Passez à la taille suivante uniquement lorsque vous êtes à l'aise avec la précédente. Avancez à votre rythme.

5. Retirez lentement, lavez, rincez et laissez sécher à l'air libre. Rangez dans un endroit frais et sec, à l'abri de la lumière directe.

Entretien

Nettoyez le dilateur anal avec le nettoyant Steri-clean avant et après chaque utilisation.


Garantie

Pour toute information concernant la garantie de votre produit, veuillez consulter notre site internet : **floravi.com**

N'hésitez pas à nous écrire pour nous faire part de vos commentaires : **info@floravi.com**

Floravi

ANAL DILATOR
DILATEUR ANAL

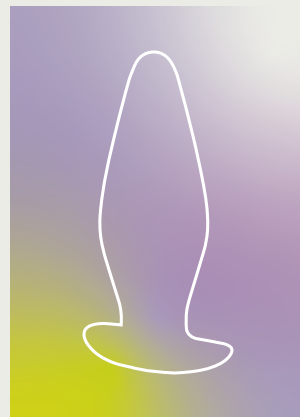

floravi.com f @ / floraviCanada

User guide

Guide d'utilisation

The anal dilator is designed to promote both sexual well-being and sensory exploration.

Instruction for use

1. Wash the dilator with warm water and Steri Clean before and after each use.
2. Apply a water-based lubricant to the dilator and around the anal area.
3. Make yourself comfortable in a quiet space. If using a set, start with the smallest size. Gently insert while making slight circular movements, without forcing. Breathe deeply and allow your muscles to relax.
4. Leave the dilator in place for a few minutes, moving it slightly if desired. Only move to the next size when you are comfortable with the previous one. Proceed at your own pace.

5. Remove slowly, wash, rinse, and allow to air dry. Store in a cool, dry place away from direct sunlight.

Maintenance

Clean the anal dilator with Steri-Clean before and after each use.

Warranty

For more information about your product's warranty, please visit our website: **floravi.com**.

For any questions or feedback, contact us at: **info@floravi.com**

Anal Dilator

Compatible with a mini massager (size S to XXL)

100% body-safe silicone

Flared shape for comfortable insertion

Smooth & flexible

Relieves tension

Dilatateur anal

Compatible avec un mini vibromasseur (grandeur S à XXL)

Silicone hypoallergénique

Forme évasé pour une insertion confortable

Surface lisse et flexible

Soulage les tensions