Ensemble de dilatateurs vaginaux

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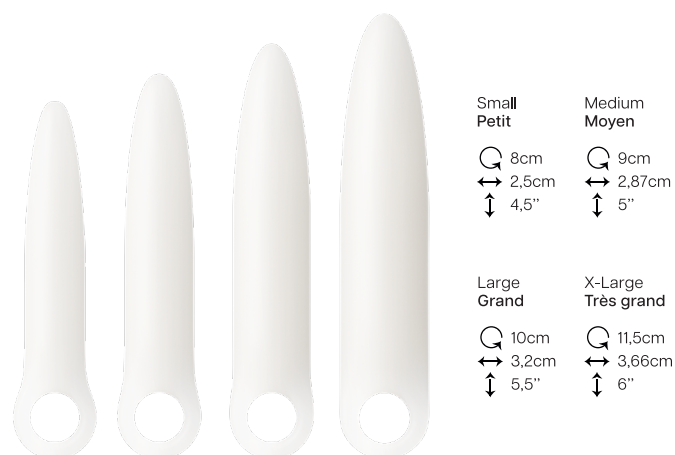
Dilator Set

Presentation

Floravi vaginal dilators help improving comfort during sexual intercourse, tampon insertion, or gynecological exams. Suitable for use independently or with professional guidance, they come in multiple sizes to progress at your own pace. Their soft silicone ensures maximum comfort.

Benefits

Support for specific conditions : Suitable for women with vulvodynia, pudendal neuralgia, vestibulodynia, dyspareunia, vaginal agenesis, or MRKH syndrome.

Personalized progression and comfort : Offer multiple sizes to progress at your own pace, with a soft silicone texture and the recommended use of Floravi's water-based lubricant for maximum comfort.

Complement : Can be used alone or alongside medical or therapeutic follow-up to enhance overall intimate well-being.

Characteristics

- Gradual sizes and diameters
- Ring-shaped base for easy handling
- Tapered tip to ease insertion
- Firm body-safe silicone but flexible

Sold for novelty use only.

Floravi  

For more information,
visit our website floravi.com

Use

Floravi dilators are used according to each woman's pace and comfort. It is recommended to practice daily exercises or at least 2 to 3 times per week, gradually progressing through sizes to avoid discomfort or worsening symptoms. Insertion should be done in a relaxed state, with the dilator pre-warmed and lubricated, in the most comfortable position. Each session begins with gentle adaptation to the dilator, using light and gradual movements. When moving to a larger size, it's important to temporarily return to the previous size to ensure optimal relaxation. Consistency and perseverance are key for effective results.

How to use

Use Floravi's water-based lubricant to make insertion easier and ensure optimal comfort. Clean the dilators before and after each use with Floravi's Steri Clean cleanser for safe and hygienic use.

If you notice bleeding, stop using immediately and consult a health professional;

Set of 4 dilators