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Floravi

RECTAL CLEANSING BULB
POIRE RECTALE

User guide

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With its ergonomic design and soft silicone texture, the rectal cleansing bulb allows for gentle and safe intimate cleansing. Its nozzle ensures easy insertion and optimal comfort, while the squeezable bulb offers simple control of the water flow for a pleasant and hygienic experience.

Instructions for use

Preparation : Fill the bulb with lukewarm water – avoid water that's too hot or too cold to ensure comfort. Check that the nozzle is firmly attached and clean before each use.

Lubrication : Apply a water-based lubricant to the nozzle and around the anus to allow for smooth and comfortable insertion.

Position : Choose a comfortable position: squatting, sitting on the toilet, or standing with one leg slightly raised. Relax your muscles before gently inserting the nozzle.

Usage : Insert the nozzle slowly and carefully. Gently squeeze the bulb to release the water. Never force it, and stop immediately if you feel any discomfort.

Hold the water for a few seconds, then expel it while seated on the toilet. Repeat 1 to 3 times if necessary, until the water runs clear.

After Use : Wash the bulb and nozzle thoroughly with warm, soapy water, rinse, and let air dry. You may also use a gentle disinfectant or toy cleaner such as Stéri-Clean for optimal hygiene. Avoid using the douche more than once per day to protect the natural flora and prevent irritation.

Additional Tip : Wait at least 30–60 minutes before anal play, allowing any residual water to be fully expelled.

Warranty

For more information about your product's warranty, please visit our website: **floravi.com**.

For any questions or feedback, contact us at: **info@floravi.com**.

floravi.com f @ / floraviCanada

Rectal Cleansing Bulb

Capacity of 150 ml

Anti-backflow design

Comfortable tip

Easy-to-insert

Soft texture

Poire rectale

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Design anti refluxement

Embout confortable

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Texture douce