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# Vaginal Weights

## Presentation

The Floravi Kegel exercise set includes four interchangeable beads with three different weights, allowing you to progress at your own pace. Prioritize your intimate well-being!

## Benefits

**Increased pleasure :** Enhance orgasmic sensations, as these muscle contractions are at the core of orgasm.

**Overall intimate well-being :** Supports overall sexual, boosts body confidence, and encourages a fulfilling intimate life.

**Target audience:** Women approaching perimenopause or menopause, those in the postpartum period, or those looking to strengthen their pelvic floor.

## Use

Insert Floravi's vaginal weights like a tampon, in whatever position feels most comfortable for you (squatting, sitting, or lying on your back). Start by wearing them for about 30 minutes a day to get used to them, then gradually increase to a few hours, at least 4 days a week. Practice pelvic floor contractions and, once comfortable, wear them during more dynamic activities (walking, household chores, squats).


Progress at your own pace by switching to heavier weights as you feel ready, from 30 g to 90 g. Regular use will help you achieve optimal results.

## How to use

If needed, use Floravi's water-based lubricant to make insertion easier and ensure optimal comfort. Clean the weights and the strap separately before and after each use with Floravi's Steri Clean cleanser for safe and hygienic use.

## Characteristics

- 4 beads (2x 30 g, 1x 40 g, 1x 50 g)
- 29 mm diameter
- Strap to facilitate removal
- Beads containing free weights working with natural bodily movements
- Body-safe silicone
- Comfortable and discreet

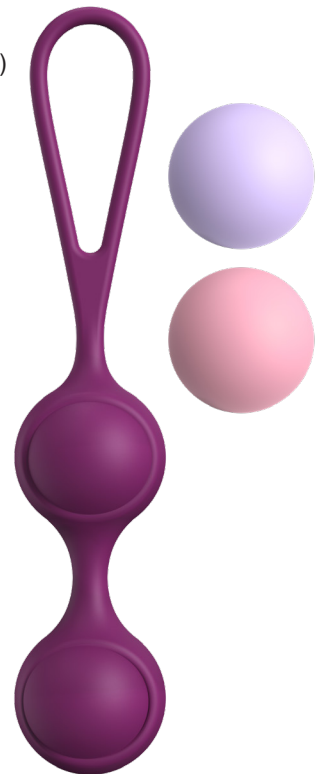*Sold for novelty use only.*

**Floravi**  

For more information,  
visit our website [floravi.com](https://www.floravi.com)